



Characterizing the Trusana Complete Arch Using GC Optiglaze™

Process Steps

- 1 Begin with a printed, fully cured Trusana denture.
- 2 To prepare Trusana denture surface to accept GC Optiglaze, roughen and clean the surface by sandblasting, pumicing, or similar.
- 3 Shake bottle well before using.
- 4 Use the paintbrush provided in the Optiglaze kit.
- 5 Using the pink tint, paint multiple coats of glaze on the denture base portion of the denture. You must light cure between each coat.
- 6 Build to the desired pink color, then highlight with other colored tints as needed.
- 7 Once final desired color is reached, paint Optiglaze Clear over entire denture, covering all surfaces, including teeth to seal and create a brilliant shine.
- 8 No additional polishing is necessary.

When using GC Optiglaze, always refer to manufacturer's instructions for use and handling.