

# Patient Care Card

## Congratulations!

Your denture has been custom made for you by dental professionals using Myerson's products. Myerson has been making medical devices for over 100 years so you can take confidence in the quality.

### **With proper care, your appliance will ensure a brighter smile.**

- Soak in water or denture cleanser when not in use to maintain pliability, shape, and to prevent drying.
- Clean daily with a soft-bristle toothbrush and mild toothpaste, soap or dishwashing liquid.
- Staining may occur from some foods. Use over-the-counter denture cleansers to remove stains.
- Handle with care to avoid drops.
- Commercial denture cleansers for removable appliances come as tablets, creams, pastes, gels, and solutions. Follow manufacturer instructions after rinsing away food particles and removing denture adhesive.
- Avoid hot water to prevent warping.
- Do not chew ice or bones while wearing dentures.
- Never use bleach, abrasive toothpaste, non-denture-specific cleaners, or a stiff brush as they can damage the appliance.
- Do not soak in alcohol-based liquids (e.g., Listerine). These may cause irritation.
- Do not use denture cleansers while the appliance is in your mouth.
- For fixed implant dentures, contact your dentist for cleaning and maintenance.

### **Contact your dentist if there are changes to your bite, or if the denture feels loose.**

*Per the EU MDR, any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user is established.*

**myerson**®